

# Adlite™ Pro

## Instructions for Use




Thank you for choosing an ADC® Adlite™ Pro, a precision-crafted LED penlight. The Adlite Pro body is machined from brass then electroplated for a durable finish. The long-lasting LED lamp is guaranteed for 10,000 hours of use.




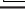
### Device Description and Intended Use

This device is intended for general, non-contact illumination.

### Symbol Definitions

The following symbols are associated with your Adlite Pro.

Symbol	Definition
	Important warning/caution
	Not made with natural rubber latex
	Phthalate free

Symbol	Definition
	Consult instructions for use
	Medical device
	Manufacturer's information
	Batch code

### General Warnings

A warning statement in this manual identifies a condition or practice which, if not corrected or discontinued immediately, could lead to patient injury, illness, or death.

**⚠ WARNING:** This instrument is NOT to be used in the presence of flammable anesthetics.

### How to Use

To turn on, use thumb to slide the textured blue power switch away from the body of the penlight. To turn off, slide switch back towards the tip.

### Battery Replacement

Adlite Pro uses 2 AAA alkaline batteries. To replace, twist LED lamp tip counterclockwise. Replace both batteries from barrel, being careful to observe correct polarity, by inserting (-) negative ends in first. Reassemble lamp tip by twisting clockwise until tight.



### Limited Warranty

ADC warrants its products against defects in materials and workmanship under normal use and service as follows:

1. LED lamp is warranted for life.
2. Adlite Pro penlight is warranted for 2 years.
3. Warranty service extends to the original retail purchaser only and commences with the date of delivery.

**What Is Covered:** Replacement of parts and labor.

**What Is Not Covered:** Transportation charges to ADC. Batteries where supplied. Damages caused by abuse, misuse, accident, or negligence. Incidental, special, or consequential damages. Some states do not allow the exclusion or limitation of incidental, special, or consequential damages, so this limitation may not apply to you.

**To Obtain Warranty Service:** Send item(s) postage paid to ADC, Attn: Repair Dept., 55 Commerce Dr., Hauppauge, NY 11788. Please include your name and address, phone no., proof of purchase, and a brief note explaining the problem.


**Implied Warranty:** Any implied warranty shall be limited in duration to the terms of this warranty and in no case beyond the original selling price (except where prohibited by law). This warranty gives you specific legal rights and you may have other rights which vary from state to state.

To register your product visit us at: **[www.adctoday.com/register](http://www.adctoday.com/register)**

This manual is available online in a variety of languages, follow the links for language options: **[www.adctoday.com/care](http://www.adctoday.com/care)**

FOR QUESTIONS, COMMENTS, OR SUGGESTIONS CALL TOLL FREE:  
**1-800-ADC-2670 OR VISIT: [www.adctoday.com/feedback](http://www.adctoday.com/feedback)**



 ADC  
55 Commerce Drive  
Hauppauge, NY 11788  
U.S.A.

Inspected in the U.S.A.  
Made in China  
tel: 631-273-9600  
toll free: 1-800-232-2670  
fax: 631-273-9659

**[www.adctoday.com](http://www.adctoday.com)**  
**[info@adctoday.com](mailto:info@adctoday.com)**

